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100%
RECYCLABLE &
VEGETAL BOTTLE

**UMONIUM³⁸
MEDICAL SPRAY**

DISINFECTANT, CLEANER SPRAY FOR MEDICAL DEVICES
DISINFECTS AS SOON AS THE FIRST MINUTE

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pH7

MEDICAL DEVICE Class IIb

BACTERICIDAL, MYCOBACTERICIDAL, FUNGICIDAL, VIRUCIDAL
Complies with: EN1040:2005, EN1275:2005, EN13727:2003, EN13624:2004, EN14348:2005, EN14476:2013, EN14561:2006, EN14562:2006, EN14563:2009

laboratoire HUCKERT'S INTERNATIONAL
20 Avenue Lavoisier
1300 Wavre - BELGIUM
Tel: +32 (0)67 89 41 00
Fax: +32 (0)67 84 37 67
www.huckerts.net

INDICATIONS: Disinfecting and decontamination for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. Transport of solid instruments. **INSTRUCTIONS:** Transport: Set the spray to the "foam" position and cover the device with the foam. Make sure to cover it well to limit evaporation. **Cleaning/Pre-disinfection:** Set the spray to the "foam" position. Spray and scrub the device to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. **Disinfection:** Set the spray to the "fine spray" position. Spray over clean devices. Ensure a minimum contact time of 5 minutes. If you are cleaning invasive medical devices: Rinse with water*, if you are cleaning non-invasive medical device surfaces: Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. *Clean drinking water at room temperature. **PRECAUTIONS:** Handle in accordance with good industrial hygiene and safety procedures. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Consult specific, directions and manual, safety data sheet.

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5 425 037 87 11524 1l e 35,2 oz.fl.

COMPOSITION UMONIUM³⁸ MEDICAL SPRAY: (A), N-benzyl-N-dodecyl-N,N-dimethylammonium chloride/N-benzyl-N,N-dimethyl-N-tetradecylammonium chloride. 3.3 g/L

laboratoire HUCKERT'S
LABORATOIRE HUCKERT'S INTERNATIONAL
20 Avenue Lavoisier
1300 Wavre - BELGIUM
Tel: +32 (0)67 89 41 00
Fax: +32 (0)67 84 37 67
www.huckerts.net

ET/00209 - Ed. 5.1 - 11/2019