



# Curatick

Oral Wound Home Care Solution

**Protect wounds  
in your mouth**



**TBM**

An adhesive transparent medical device for cuts in your mouth, prevents cuts in your mouth from being contaminated and protecting such wound.




## Oral Wound Home Care Solution



Attach it to an wound in your mouth.

**'Tick'**

Attach this brand new wound protective patch in the mouths of children to adults.

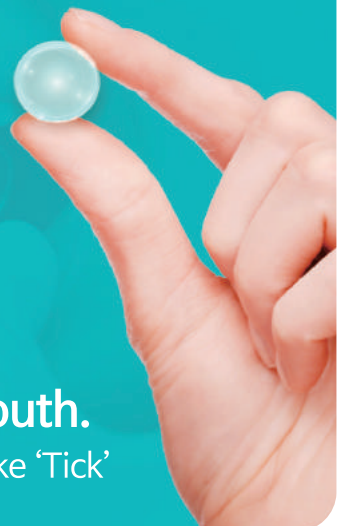

Now, just **'Tick!'** for an wound in your mouth.

There's no need sear, apply, or rinse. Just simply attach it. like 'Tick'

# A protective patch for an wound in your mouth which is applicable for a variety of wounds.



## A protective patch for an wound in your mouth

It protects the wound by preventing it from coming in contact with anything else.



2~4 Hours

## Attachable for 4 hours

It can be attached for 2 to 4 hours.  
(The time may vary according to the oral conditions)



Wound

## Low feeling of irritation/ rapid adhesion

Easy, fast adhesion and application  
Easy to carry around thanks to the form of strip



## Water-soluble adhesive patch

It is not adhesive in the ordinary times,  
but when it comes in contact with water/  
saliva, it becomes gel with adhesion.



## The adhesive surface is naturally removed

When the adhesive surface melts thanks to reaction with saliva in your mouth, the protective surface is automatically removed.






## Raw materials that meet the specifications

Made by using raw materials that satisfy the specifications of KP and USP

✔ **Protects the wound in your mouth from foods, germs, tongue, and tobacco smoke.**

- ✔ Protects the wound by preventing external irritants from coming in contact with the wound
- ✔ Easily attached to the wound!
- ✔ It sticks to the wound for up to four hours. (The time may vary according to your oral conditions.)
- ✔ Protects the wound and mucous membrane in your mouth
- ✔ Increased freshness thanks to mint flavor
- ✔ Containing vitamin E

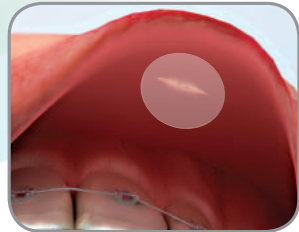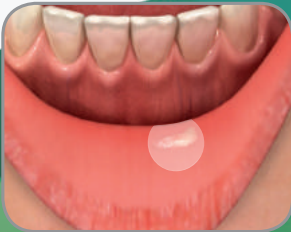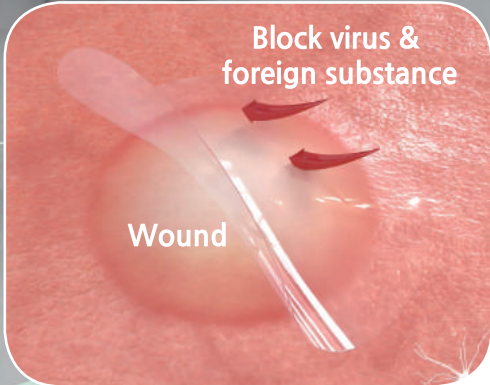

✔ **Product Strong Point - 3N**

**No-Steroid**

It does not contain steroid. Steroids are corticosteroid hormones that can cause serious side effects in infants and adults.

**No-Pain**




The flexible material protects the cuts and relieves pain in the mouth by forming physical protective film on the affected area.

**Natural Cure**

It improves the self-regeneration of oral mucosa by blocking the affected area from external stimuli for long time(2~4hrs).



## Instructions

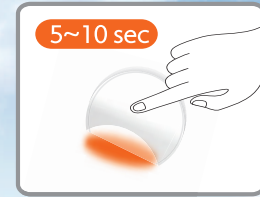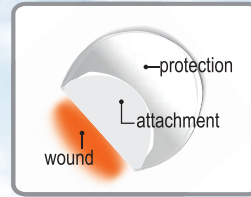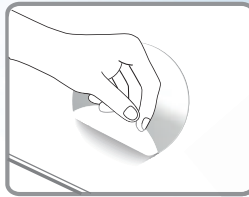

1. Clean the affected area and wipe off if there's a lot of saliva (too much saliva interferes with adhesion)
2. Pull off the product from the release liner
3. Apply adhesive layer on the wound
4. Press slightly for 5 seconds until adhesion occurs



## Cautions



This medical product is for single use. If reused, the product may cause infection, cross infection, and inflammation.

1. Do not use products that have expired, damaged, or contaminated.
2. If you experience any swelling, allergic reaction, upset stomach or any signs of infection, consult with your doctor.
3. Do not use if an infection is suspected and if an infection occurs (a wound that appears to be abnormal, such as fever or edema.)
4. Do not use on deep wounds with excessive bleeding.
5. If the product is attached and removed forcibly, the wound may come off. Do not remove it.
6. Remove the dropped protection layer from a wound.
7. Sufficient care is required for sleeping or pediatric patients to remove the protection layer.
8. The time of attachment may be reduced when stimulated by tongue or food.



## Curatlick Application

### Case#1,2

Contusion



Inside the Lip




### Case#3,4

Gum




Palate

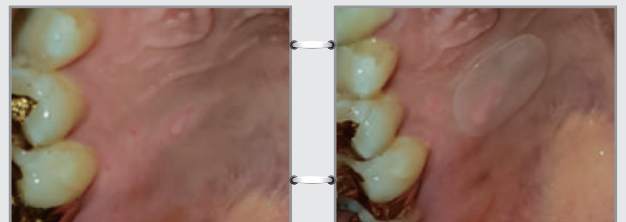

### Case#5,6

Tongue

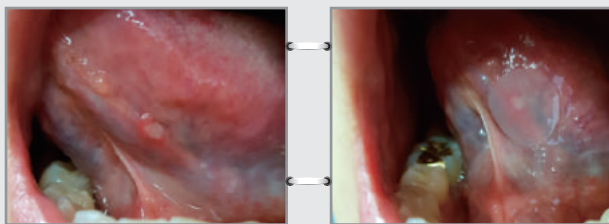

Buccal mucosa

