

CY PREP PAP TEST-CERVIX-BRÚM®

For Collection of Cervical Specimen

Art No : 6888-0300

Intended Use

CY PREP PAP TEST-CERVIX-BRÚM® is designed for use in cell collection for Cervical Cytology. It is a broom like cervical cell sampling device for simultaneous collection of both the ectocervical and endocervical cells.

Contents :

300 CY PREP PAP TEST-CERVIX-BRÚM® cervical sampling device

- For use with liquid based cytology
- For use with conventional pap smear

Storage :

Room temperature & dry condition. Keep away from moisture.

Quick Reference Guide

1. Contact the cervix with the CY PREP PAP TEST-CERVIX-BRÚM® and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix. While maintaining pressure in the direction of the cervix, gently rotate the CY PREP PAP TEST-CERVIX-BRÚM® 5 times in a clockwise direction by rolling the stem between the thumb and the forefinger.

Choose the preferred method

(A) Conventional Pap Smear

2A. Transfer the cells collected to glass slide with a painting action, applying first one side of the bristles and then the other side.

3A. Immediately apply fixative to the slide and label. Place the slide in slidemailer.

(B) Liquid Cytology

2B. Insert the Cervix Brum into the vial with the preservative solution and follow the instructions as given by the Liquid Cytology manufacturer.

3B. Reseal and label the vial.

4, Complete the patient requisition form and send together with the slidemailer or vial to the laboratory.

Precautions & Warning :

- Do not use on pregnant women.
- Single use only
- Dispose of this device in accordance with the local medical hazardous waste practice.



Manufactured by:
FJORD DIAGNOSTICS
Puchong 47100 SGR. M'SIA.
www.fjorddiagnostics.com

STERILE EO

UNLESS PACK IS OPENED OR DAMAGED

SYSTEM CERTIFIED

ISO 13485:2003 No. 00095/0