




UASure II

Blood Uric Acid Monitoring System

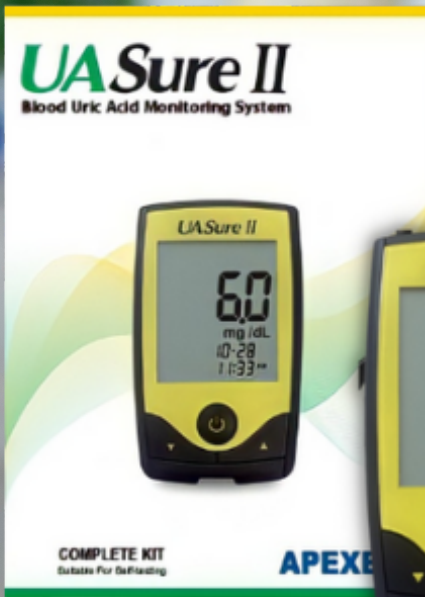

NIE AKL 20101221082



Hct 20% - 60% Sampel darah 1,5 µL Waktu uji singkat Dimensi alat mungil Terkoneksi PC

SAYANGI KELUARGA DENGAN RUTIN PERIKSA ASAM URAT

Importir dan Distributor Resmi:

PT ISOTEKINDO INTERTAMA

Jl. Raya Kebayoran Lama 309-C, Jakarta Selatan, DKI Jakarta, 12210 | (021) 5305073-74 | 0811.171.0055

info@isotekindo.co.id | www.isotekindo.co.id | isotekindo.intertama | @isotekindo



linktr.ee/isotekindo

UASure II

Blood Uric Acid Monitoring System

CARA PENGGUNAAN



Langkah 1
Pasang tes strip ke dalam alat.



Langkah 3
Tempelkan jari ke bagian sensor, layar akan menampilkan bar timing "----"



Langkah 2
Pijat jari dengan lembut, tusuk menggunakan lancet.




Langkah 4
Hasil akan diperoleh sekitar 45 detik.


SPESIFIKASI ALAT

Parameter	Keterangan
Kisaran Uji	3-20 mg/dL (179-1190 μ mol/L)
Volume Sampel	$\geq 1.5 \mu$ L
Kisaran Hematokrit	20 - 60%
Tipe Tampilan	Layar LCD
Memori	200 Hasil tes
Waktu Pengukuran	10 detik
Ukuran	80x 48x13 (mm)
Berat	34 g
Baterai	1 x CR 2032 3V Lithium coin cell battery
Daya tahan baterai	1.000 tes penggunaan
Data Transfer	USB

UJI AKURASI SAMPEL ARTERI DAN VENA

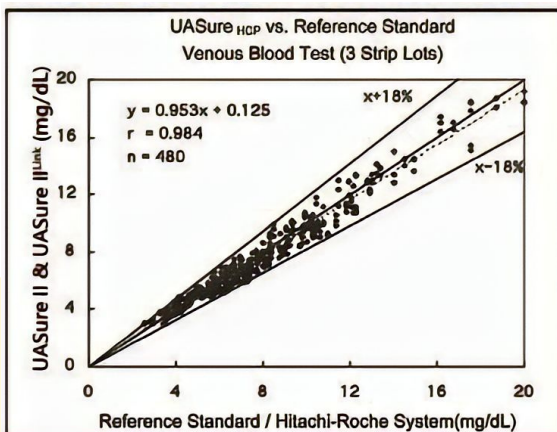

Clinical Site Studies with Venous Blood Sample	
Number of Sample:	480
Range:	2.6 mg/dL to 20.2 mg/dL
Slope:	0.953
Intercept:	0.125 mg/dL
Correlation Coefficient:	0.984

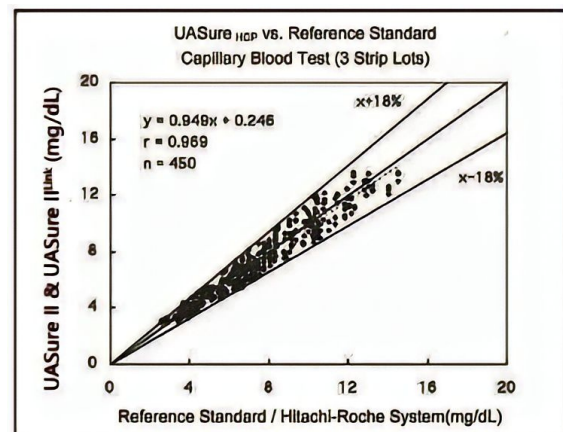

Clinical Site Studies with Capillary Blood Sample	
Number of Sample:	450
Range:	2.6 mg/dL to 14.5 mg/dL
Slope:	0.949
Intercept:	0.246 mg/dL
Correlation Coefficient:	0.969